GAMINIO NAUDOJIMO INSTRUKCIJA USER'S MANUAL

FIXALTO®

JUOSMENS ĮTVARAI BŪSIMAI MAMAI KVP 6-6-1 / KVP 6-6 (SU PAPILDOMU DIRŽU*) LUMBAR ORTHOSES (MATERNITY BELTS) KVP 6-6-1 / KVP 6-6 (WITH AN ADDITIONAL BELT*)

SoftFIX® Lumbo Medium MAMA / SoftFIX® Lumbo Strong MAMA

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Anatomically formed knitted orthosis gently provides support for the abdomen and reduces the pressure directed to the waist (back) during pregnancy; equally allocates the weight along the back and helps to avoid the back (spinal) and waist pains. Helps to sustain the correct set-up and to move actively. Protects from the muscles overstrain. Orthosis is elastic, breathable and comfortable to wear.

* additional belt is using for adjusting the strenght of the reduction. It can be knocked out.

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Size. If you want the product to perform its functions effectively, it is necessary to choose the right size (see Table below). A correctly chosen orthosis should not slip down, strangle or cause any other unfavourable sensations.

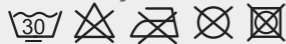
	S	M	L	XL	XXL
cm	90-100	100-110	110-120	120-130	130-140

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Recommendations: It is recommended to wear the belt lying on your back. Do not wear the orthosis if you notice any skin sensitiveness.

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It is recommended to wash the orthosis in water not warmer than 30°C. Use mild laundry detergents. Dry at ambient temperature away from heating appliances and direct sun. Keep away from fire and heat. If the product is fitted with a Velcro, it should be washed and dried closed. If you wash in a washing machine, place the orthosis into a special bag to prevent the Velcro from spoiling other clothes.

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The warranty is provided in accordance with the procedure established by the laws. The warranty obligations shall be observed provided that during the said period the product is used for its direct purpose and its construction is not modified.

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Thank you for choosing our product. We trust you'll feel better.

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