KELIO ĮTVARAS KA 3-3-1-2 / KNEE ORTHOSIS KA 3-3-1-2

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Recommended: during the rehabilitation period after knee surgery, fractures, ligaments, tendons, muscle sprains or tears, and in case of various internal injuries or injuries of the knee or meniscus; bursitis. The orthosis is designed to limit the lateral mobility of the knee joint. Aluminum hinges distribute the load on the knee joint evenly. The orthosis is made of elastic air-permeable fabric with a warming effect. Reduces pain.

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Composition: 75% elastic polyurethane, 12.5% polyamide, 12.5% viscose.

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Size. If you want the product to perform its functions effectively, it is necessary to choose the right size (see Table below). A correctly chosen orthosis should not slip down, strangle or cause any other unfavourable sensations.

	S	M	L	XL	XXL
cm	33-36	36-40	40-44	44-46	46-50

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Directions for use:

1. Place the orthosis on knee joint position where the split with the pad fix the kneecap.
2. Fasten the size-adjusting strips.
3. Fasten the reinforcing straps.

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Recommendations: Consult your doctor on the expediency of wearing the orthosis and the recommended time of wearing it. Do not wear the orthosis if you notice any skin sensitiveness.

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It is recommended to hand-wash the orthosis in water not warmer than 30°C. Use mild laundry detergents. Be careful to take out all hard details/parts before washing. If the product is fitted with velcro, it should be washed and dried closed. If you wash in a washing machine, place the orthosis into a special bag to prevent the Velcro from spoiling other clothes.

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The warranty is provided in accordance with the procedure established by the laws. The warranty obligations shall be observed provided that during the said period the product is used for its direct purpose and its construction is not modified.

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Thank you for choosing our product. We trust you'll feel better.

Gamintojas:

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Tel.: (8-37) 313303

el. paštas: info@ortopedija.lt

www.ortopedija.lt

Producer:

Partizanų str. 17, LT-49476 Kaunas, Lithuania

Tel.: +370 37 313303

e-mail: info@orthopedic-pro.com

www.orthopedic-pro.com

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Informacijos parengimo data 2020-07-01