GAMINIO NAUDOJIMO INSTRUKCIJA

USER'S MANUAL

JUOSMENS – KRYŽKAULIO ĮTVARAS KR 0-26 / LUMBOSACRAL ORTHOSIS KR 0-26

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Provides a strong degree of fixation of the lumbosacral spine. Limits the sharp movements. With metal modeled stiffening ribs and additional adjustable straps.

Recommended wear: after traumas and operations; in case of pain in the lower back due to the pain, aggravation of osteochondrosis; to save the achieved a therapeutic effect in the course of manual therapy and medical gymnastics; for the prevention of complications in the treatment of diseases of the lumbar spine.

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Size: In order the product performs its functions effectively correct size has to be chosen (see Table below). Orthosis should not slip down, strangle or cause any other unfavourable sensations.

Dydis / Size	XS	S	M	L	XL	XXL	ūgis/ height
a+b/2, cm	<70	70-85	85-97	97-105	105-115	>115	>165

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Recommendations: consult your therapist on the expedience and duration of wearing the product, especially in cases of skin damages in the respective area of the body, bloodstream disorders, heart deficiency. The waist area shall not be covered with oil, lotions or other greasy preparations before putting on the orthosis. Do not wear the orthosis in case of any allergic skin changes or other unpleasant sensations.

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The warranty is provided in accordance with the procedure established by the laws. The warranty obligations shall be observed provided that during the said period the product is used for its direct purpose and its construction is not modified.

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Thank you for choosing our product. We trust you'll feel better.

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