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Size selection and use of the orthosis:

The size of the SpecialFIX® Patella Strap is universal.

1. Place the strap on the knee joint so that the pad A is under the patella B (see Picture).
2. Tighten both straps simultaneously so that the pelota stays in place and does not move sideways (see Picture).
3. Fasten the velcro strip by choosing a compression of the strap to prevent the blood flow disturbance, but keeping the strap in place.

Maintenance:

SpecialFIX® Patella Strap orthosis is hand or machine washable (using gentle wash cycle at temperatures up to 30°C and mild detergents). Please do not use fabric softeners or aggressive products (products containing chlorine). To protect the surface of the material from damage caused by other laundry, wash it separately in a laundry bag or net. Allow to air dry, do not expose to direct heat, e.g. from the sun, an oven or radiator.

Warranty

The product has been created with great care, under constant monitoring and quality check.
The product is covered by warranty according to the procedure prescribed by laws. Warranty obligations are fulfilled when the product is used as intended and its design has not been changed. Please keep the purchase document. If you have any complaints concerning quality, please contact the store where you purchased the product (for hygiene reasons the product should be returned in a clean condition after it has been washed).

We declare that the product meets the safety requirements of technical specifications applicable to medical devices (equipment) and is safe if used as intended.

Disposal

Please dispose of the product in an environmentally-friendly way by discarding it into the container for textile waste. The product package is suitable for recycling.

The information contained herein was prepared on September 30, 2020

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EN Instruction for use

Thank you for choosing our product. We would like to assure you that our SpecialFIX® Patella Strap meets the highest standards of quality and safety.

Before you start using the product please read this instruction as it provides important information about the intended use of the product, indications and relative contraindications, handling and disposal. Please keep this instruction leaflet for future reference. If you need more information about the product, please do not hesitate to contact the specialists at our sales locations or write us to info@orthopedic-pro.com.

Purpose:

SpecialFIX® Patella Strap is a universal size knee strap designed to be worn for knee pain.

The strap is made of air-permeable fabric, without restricting movements and reducing the risk of injury. The anatomically shaped strap pelota adapts well to the knee joint and maintains the patella in place, reducing the load on the patellar ligament and providing stability for active movement of the entire knee joint. The pelota stimulates infrapatellar (postpartum) pressure to relieve pain. Therefore, the strap allows to return to favorite activities: football, basketball, running or skiing and not feeling pain when climbing the stairs.

The orthosis enhances joint stability and neuromuscular stimulation to restore normal functional movement.

Thanks to the velcro strip used for fastening, the strap can be easily put on by choosing the required volume.

The strap is designed to be worn during active movement.

Indications (symptoms when splints are recommended):

- osteochondropathy of tibial bone roughness (Osgood-Schlatter disease),
- patellar ligament sprain,
- patellofemoral (patella and femur) osteoarthritis,
- injury prevention.

Conditional contraindications

(symptoms in which the splint cannot be used without consulting a specialist):

- skin lesions (damage to integrity, rash, allergies, etc.) in the part of the leg covered by the strap,
- circulatory or hypersensitivity disorders (e.g. due to diabetes mellitus),
- swelling of unknown origin.

Safety links:

- The product is designed for personal use.
- Consult a doctor about the purpose and duration of wearing the product.
- Also consult a doctor if the ailments or pain do not decrease or even increase while wearing the product.
- If you need to wear (use/wear) other compression products (e.g. compression socks) due to your medical condition, consult your doctor to see if a strap can be worn with them.
- Do not wear the strap if you have allergic skin lesions or other unpleasant sensations. In this case, we recommend consulting your doctor.
- Allergic skin reactions are individual and there is no reason to return the product. Pay attention to the composition of the material from which the product is made!
- The product does not contain latex.
- Use the strap while moving. If you want to use the strap at rest, talk to your doctor.
- It is not advisable to apply ointments, lotions or other greasy preparations to the knee joint area before putting on the strap.
- Wash the product regularly.
- Protect the product from direct sunlight, fire and heat.