Thank you for choosing our product. We would like assure you that your foot insoles meet the highest standards of quality and safety.

Before you start using the product please read this instruction as it provides important information about the intended use of the product, indications and relative contraindications, use, care, safety instructions and disposal. Please keep this instruction leaflet for future reference. If you need more information about the product, please do not hesitate to contact the specialists at our sales locations or write us to info@orthopedic-pro.com.

Purpose and principle:

Multifunctional foot insoles *StepFIX™ Active* are medical devices. They are designed to be worn preventively or with the presence of any of various foot disorders while allowing for an active lifestyle or recreational sports activities. These insoles provide a firm, semi-rigid support for the longitudinal arch, evenly distributing pressure across the entire surface of the foot. They can protect against developing foot pain or alleviate existing discomfort. Depending on the type and purpose of the insoles, they may restrict the movement of certain metatarsals or their joints. The insoles feature a cushioning layer that absorbs shocks. This cushioning layer is made from sustainable (recycled) material, characterised by high wear resistance and no lasting memory effect, meaning it does not change due to weight impact and returns to its original form. The heel pocket stabilises the position of the calcaneus, enhancing protection against accidental traumatic movements to the ankle joint. *StepFIX™ Active* insoles are intended for activewear, hence, they do not have a transverse arch lift.

StepFIX™ Active Basic is ultra-thin in the toe area. Although recommended for active movement and running, they are suitable for preventive use even without foot disorders.

StepFIX™ Active Lateral is recommended for recreational engagement in sports such as tennis, squash, badminton, table tennis, and jogging. These insoles stabilise the soles of worn footwear, and promote the movement of the big toe while stabilising the other toes. By directing the front part of the foot inwards (pronation), they reduce the likelihood of ankle joint injury during take-off and landing moments.

StepFIX™ Active Bilateral is suitable for recreational activities like mountain hiking and Nordic walking. These insoles stabilise the soles of worn footwear and feature unique stabilisation of the first and fifth metatarsal heads, supporting and restricting the movement of these metatarsal joints without hindering walking.

StepFIX™ Active Complete is suitable for recreational activities such as skiing, jogging, and cycling. These insoles stabilise the soles of worn footwear, limiting the range of motion in the front and middle parts of the foot, thereby eliminating excess loads on these areas.

 Use only if medically indicated or prescribed by a doctor.

Name of the medical device	Registration number issued by the State Service for Accreditation of Health Care Activities
StepFIX™ Active <i>Basic</i>	LT-CA01/MDD/1-00163/23
StepFIX™ Active <i>Lateral</i>	LT-CA01/MDD/1-00164/23
StepFIX™ Active <i>Bilateral</i>	LT-CA01/MDD/1-00165/23
StepFIX™ Active <i>Complete</i>	LT-CA01/MDD/1-00166/23

Composition:

- Upper layer: 100% polyester, padding: 100% recycled polyurethane;
- Cushioning layer: rubber;
- Stabilising components: thermoplastic copolymer / thermoplastic PE fabric.

Indications (signs when an insoles are recommended):

StepFIX™ Active Basic – for longitudinal arch or forefoot pain, bursitis, calluses, capsulitis, flat feet; for functional therapy of Freiberg infraction, metatarsalgia, Morton's neuroma and pronation.

StepFIX™ Active Lateral – for metatarsal protrusions, arthritis of the smaller toe joints, and pronation for functional therapy and prevention.

StepFIX™ Active Bilateral – for pain in the longitudinal arch; arthritis and stiffness of the big toe metatarsophalangeal joint (*Hallux rigidus*, *Hallux limitus*); diabetic foot issues; for functional therapy and prevention of Morton's neuroma, arthritis, hallux valgus, sesamoiditis, flatfoot, forefoot pain.

StepFIX™ Active Complete – for the forefoot pain, capsulitis, bursitis, metatarsalgia for functional therapy; arthritis and stiffness of the big toe metatarsophalangeal joint (*Hallux rigidus*, *Hallux limitus*), forefoot protrusions; metatarsalgia; cuboid syndrome, treatment of arthritis in the forefoot and midfoot, sesamoiditis.

 **Conditional contraindications**
(symptoms for which the insoles should not be used without consulting a specialist):

- *StepFIX™ Active Lateral* is not suitable for X-shaped foot deformities.

 **Safety icons:**

- Choose closed-type footwear of the appropriate size for safety. When purchasing footwear, measure it with the *StepFIX™ Active* foot insoles (if the existing insole of the chosen shoe model comes out, replace it with the insert, if it doesn't come out, put the insert on top).

- It usually takes 2-6 weeks for feet to get used to foot insoles. The greater the foot deformity, the stronger the pressure on specific foot points, and more time is needed for comfortable walking. Until you become accustomed to using them, avoid using the foot insoles for long distances. In the first few days, walk with them until you feel no discomfort – start with 1 hour and gradually increase the duration.

- We recommend that you consult your doctor if your discomfort or pain continues to increase or even decreases after wearing the product.

- Always use foot insoles in both shoes, unless otherwise instructed by your supervising doctor.

- Do not use foot insoles if there is damage to the integrity of the skin on the foot.

- The product is intended for personal use.

- Do not wear footwear with foot insoles without socks.

- We recommend occasionally removing the foot insoles when you take your shoes off to evaporate the accumulated moisture.

- The design of the product may not be altered independently, except for trimming the forefoot (see illustration) within 5 mm limits to adjust its shape and insoles length.

Sizes: 36-50. If you are in doubt about the size, we recommend that you choose a larger one, as the liner can be shortened up to 5 mm by trimming as shown in the picture.

Sizes	Insoles length*, cm
36	24.0
37	24.7
38	25.3
39	26.0
40	26.7
41	27.3
42	28.0
43	28.7
44	29.3
45	30.0
46	30.7
47	31.3
48	32.0
49	32.7
50	33.3

*tolerance +/- 2 mm

Maintenance:

Remove dirt with a damp cloth. Do not use chemical cleaning or bleaching agents. Dry at room temperature away from heating appliances and direct sunlight. Keep away from fire and heat.

Warranty

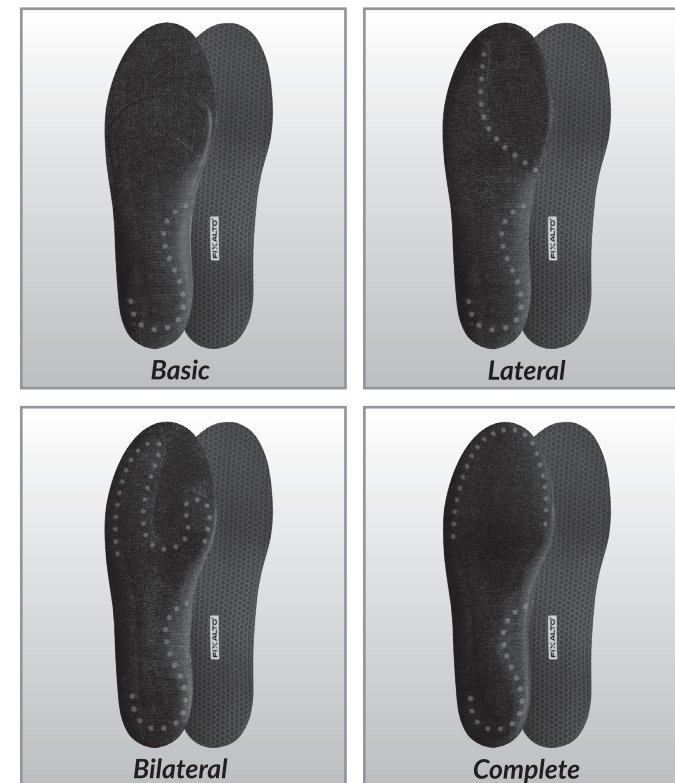
Given that the foot insoles are a wearable product, it comes with a 6-month manufacturer's warranty. Warranty obligations are fulfilled if the product is used in accordance with the care recommendations for the specified period of time and is not independently modified in design. Please note that natural wear and tear is not considered a defect and no claims are accepted. Keep the purchase receipt. In case of product quality complaints, please contact the place of purchase (for hygiene reasons, please bring a clean product, following the care recommendations in the instructions).

 **We declare that the product complies with the requirements of the Medical Device Regulation 2017/745 and is safe when used as intended.**

Disposal

Dispose of the foot insoles in textile collection containers. The packaging of the product is intended for recycling.

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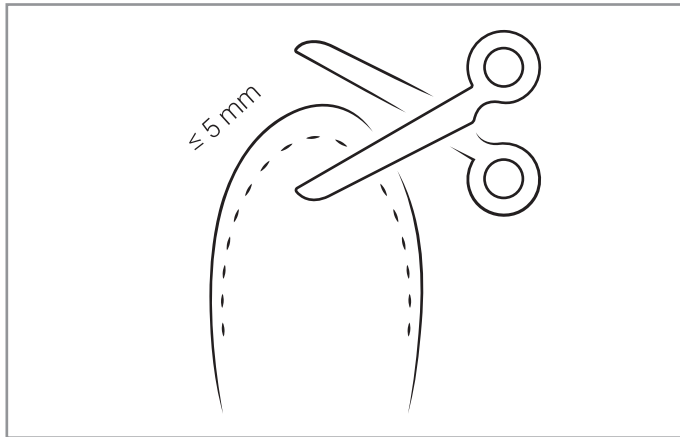
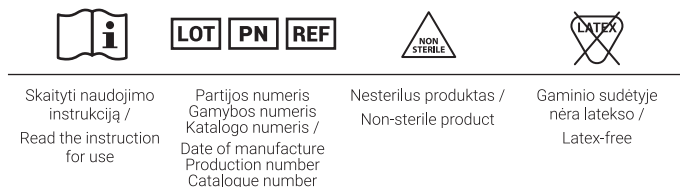
EN Symbol meanings on the product label and in the instruction for use:

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- a) gamintojui: AB „Ortopedijos technika“ Partizanų g. 17, 49476 Kaunas, Lietuva kokybe@ortopedija.lt