- the wrist area.
- Wash the product regularly.
- Keep the product away from direct sunlight, fire and heat.

Size selection and use of the orthosis:

The wrist orthosis *BasicFIX™ Carpus Light 2.0* (RT 1-8-3) has a universal size (for wrist circumferences from 13-25 cm). Designed for left or right hand.

To put on an orthosis:

- Wrap the orthosis around your wrist and, with your arm close to your body, fasten the band with the hook-and-loop fasteners (Figure 1).
- Wrap the rest of the orthosis around the wrist again in the direction of the arrow (Figure 2).
- Fix the end of the tape in the designated place with the hook fastener detail (Figure 3).

Maintenance:

Hand wash or machine wash on gentle cycle in water up to 30 °C. Choose a fabric-friendly detergent. Do not use fabric softeners. When using the washing machine, we recommend to fasten the orthosis and put it in a laundry bag to prevent the Velcro stripe from damaging other laundry. Dry at room temperature away from heating appliances and direct sunlight.

 Attention! Do not use chemical cleaners and bleaches.

Warranty

Given that the orthosis is a wearable product, it comes with a 6-month manufacturer's warranty. Warranty obligations are fulfilled if the product is used in accordance with the care recommendations for the specified period of time and is not independently modified in design. Please note that natural wear and tear is not considered a defect and no claims are accepted. Keep the purchase receipt. In case of product quality complaints, please contact the place of purchase (for hygiene reasons, please bring a clean product, following the care recommendations in the instructions).

 We declare that the product complies with the requirements of the Medical Device Regulation 2017/745 and is safe when used as intended.

Disposal

Dispose of the orthosis in textile collection containers. The packaging of the product is intended for recycling.

Report a serious incident involving the item:

- | | |
|---|---|
| a) to the manufacturer:
AB Ortopedijos technika
Partizanų g. 17, 49476 Kaunas, Lithuania
quality@ortopedija.lt | b) to the competent authority:
To the State Office for Accrediting for Health Care
Activities under the Ministry of Health
A. Juozapavičiaus g. 9, LT-09311 Vilnius, Lithuania
vaspvt@vaspvt.gov.lt |
|---|---|

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EN Instruction for use

Thank you for choosing our product. We would like assure you that our wrist orthosis *BasicFIX™ Carpus Light 2.0* (RT 1-8-3) meets the highest standards of quality and safety.

Before you start using the product please read this instruction as it provides important information about the intended use of the product, indications and relative contraindications, use, care, safety instructions and disposal. Please keep this instruction leaflet for future reference. If you need more information about the product, please do not hesitate to contact the specialists at our sales locations or write us to info@orthopedic-pro.com.

Purpose and principle:

The wrist orthosis *BasicFIX™ Carpus Light 2.0* (RT 1-8-3) is a medical device. It is recommended for carpal tunnel syndrome, sprains of ligaments and tendons.

The orthosis is made of textile fabric with a hook-and-loop strap to adjust the volume and tightness. Easily stabilizes, warms the wrist joint and reduces pain. The orthosis does not disturb hander movement, reduces the likelihood of injury recurrence.

 Use only if medically indicated or prescribed by a doctor.

Name of the medical device	Registration number issued by the State Service for Accreditation of Health Care Activities
Wrist orthosis RT 1-8-3 (BasicFIX™ Carpus Light 2.0)	LT/CA01/MDD/1-00059/07

Composition: 64 % polyester, 23 % polyamide, 13 % elastane.

Indications (symptoms for which orthosis is recommended):

- carpal tunnel syndrome, tendovaginitis;
- wrist tendon injuries (sprains, ruptures);
- for the prevention of repeated injuries.




 **Conditional contraindications** (symptoms for which the orthosis should not be used without consulting a specialist doctor):

- skin changes (integrity violations, rashes, allergies, etc.) in the part of the arm covered by the orthosis;
- swellings of unknown origin.

 **Safety icons:**

- The product is for personal use.
- Consult a doctor about the purpose and duration of wearing the product.
- Also consult a doctor if the ailments or pain do not decrease or even increase while wearing the product.
- If you need to wear other compression products (e.g. a compression sleeve) due to your health condition, you must consult with your doctor whether you can wear the orthosis together with them.
- Do not wear the orthosis if you have allergic skin changes or other unpleasant sensations. In this case, we recommend consulting a doctor.
- Allergic skin reaction is individual and is not a reason to return the product. Pay attention to the composition of the material from which the product is made!
- The product is latex free.
- Use the orthosis while moving. If you want to use orthosis while you are at rest, consult your doctor.
- Before applying the orthosis, it is not recommended to apply lotions or other oily products to

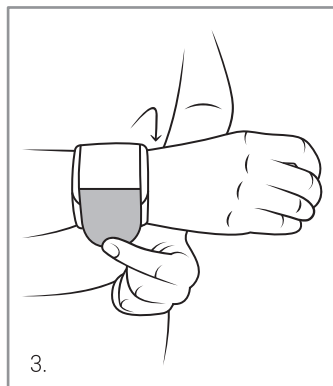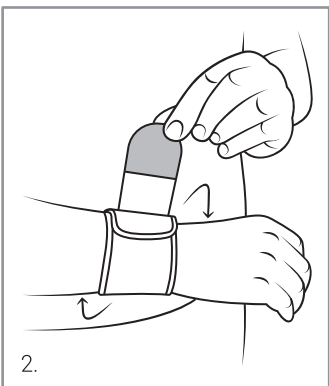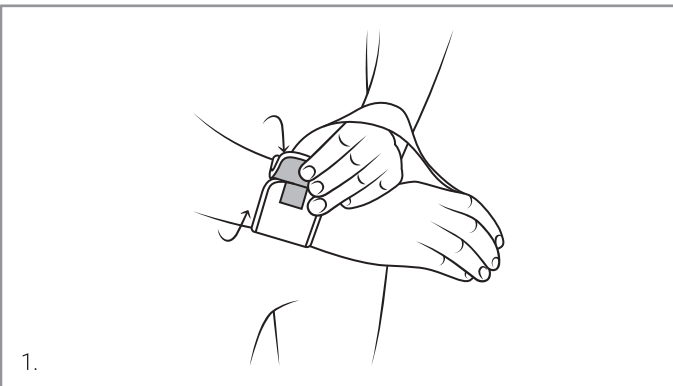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EN Symbol meanings on the product label and in the instruction for use:

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