PETIES ĮTVARAS AP 2 / SHOULDER ORTHOSIS AP 2

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Recommended for the treatment of tendon and ligament strains, bruising and inflammation of the shoulder joint. The rehabilitation period after surgery.

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Composition: 74% polyamide, 26% elastic polyurethane.

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Recommendations:

Consult your doctor on the expediency of wearing the orthosis and the recommended time of wearing it. Do not wear the orthosis if you notice any skin sensitiveness.

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It is recommended to clean with damp cloth. If it necessary, after removing foam, the cloth can be washed by hand or in water not warmer than 30°C. Use mild laundry detergents. If you wash in a washing machine, place the orthosis into a special bag. Dry at ambient temperature away from heating appliances and direct sun. Keep away from fire and heat.

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The warranty is provided in accordance with the procedure established by the laws. The warranty obligations shall be observed provided that during the said period the product is used for its direct purpose and its construction is not modified.

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Thank you for choosing our product. We trust you'll feel better.

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