- In case of discomfort, numbness, changes in skin colour or swelling during wear, stop using the support and contact your physician (skin allergic reaction is an individual condition and it is not subject to the product return).
- The support should be used during physical activity. Please consult a physician if you want to use the support during rest periods.
- Do not expose the support to ointments, lotions or other products that contain oils or acids.
- To retain compression of the knitted product, please wash the product regularly.
- Keep the product away from open flame, sun, and other sources of heat.

#### Size selection and application/removal of the support:

It is important to select an appropriate size in order to ensure the support functions (please see the table). A brace of the appropriate size should not slip down or compress the leg.

Circumference measured 14 cm above and 12 cm below the middle of the kneecap with a given flexion of 30° (see Picture 6).

- To apply the support, grasp it with both hands on the sides and on the upper end of the coil springs (see Picture 1).
- Pull the support over the knee and position the silicone knee pad so that the kneecap is surrounded without pressure (see Pictures 2, 3 and 4).
- Correct the position of the support if required.
- To remove the support, grasp it with both hands on the sides and on the lower end of the coil spring, then pull the support down over the foot (see Picture 5).

*Note: after removal of the support, temporary marks of internal structure of the product may be visible on the skin, they will disappear rapidly.*

#### Maintenance

SoftFIX® Genu knee support is hand or machine washable (using gentle wash cycle at temperatures up to 30°C and mild detergents). Please do not use fabric softeners or aggressive products (products containing chlorine). To protect the surface of the material from damage caused by other laundry, wash it separately in a laundry bag or net. Allow to air dry, do not expose to direct heat, e.g. from the sun, an oven or radiator.

#### Warranty

The product has been created with great care, under constant monitoring and quality check.

The product is covered by warranty according to the procedure prescribed for by laws. Warranty obligations are fulfilled when the product is used as intended and its design has not been changed. Please keep the purchase document. If you have any complaints concerning quality, please contact the store where you purchased the product (for hygiene reasons the product should be returned in a clean condition after it has been washed).

We declare that the product meets the safety requirements of technical specifications applicable to medical devices (equipment) and is safe if used as intended.

#### Disposal

Please dispose of the product in an environmentally-friendly way by discarding it into the container for textile waste. The product package is suitable for recycling.

The information contained herein was prepared on January 17, 2018

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Pateikiama informacija parengta 2018-01-17

## EN Instruction for use

Dear Customer,  
Thank you for choosing our product. We would like to assure you that our knee support SoftFIX® Genu meets the highest standards of quality and safety.

Before you start using the product please read this instruction as it provides important information about the intended use of the product, indications and relative contraindications, handling and disposal. Please keep this instruction leaflet for future reference. If you need more information about the product, please do not hesitate to contact the specialists at our sales locations or write us to [info@orthopedic-pro.com](mailto:info@orthopedic-pro.com)

### Intended use

SoftFIX® Genu is anatomically contoured knee support produced combining high-quality yarn with advanced horizontal knitting technique which allows to achieve 3D shaping of the fabric for an optimum fit. Flexible medial and lateral stainless steel stays keep the support from rolling or bunching, also support a natural gait. The knee support is specially designed to provide excellent compression and stabilization for weak or injured knees. A silicone insert centres the patella and leaves it pressure free. To simulate physiological taping technique different structures are used to increase functionality of the knee support: a tighter structure for ligament function support and higher elasticity areas for lighter compression of the muscles. Pressure-free graduated compression at the edges prevents constriction of blood circulation. Yarn containing silver ions has antibacterial properties. The support is exceptionally comfortable due to a large number of fibers in the yarn and better breathability.

The knee support is intended to be used during physical activities and exercise.

### Indications:

- knee pain and swelling (after surgery or trauma),
- strain injury of knee joint ligaments or tendons,
- patellar tracking disorder or patellar instability,
- degenerative disorders of the knee joint (arthrosis),
- patellar pathologies of various origins,
- Osgood-Schlatter disease,
- inflammation of knee joint,
- injury prevention.

The knee support is also recommended during the course of rehabilitation.

### Absolute contraindications - not known.

### Contraindications:

- skin diseases (rash, allergy, etc.), injuries in the area covered by the support,
- prominent scars that are swollen,
- reddening and hyperthermia of the fitted limb,
- pronounced varicose veins,
- blood circulation disorders,
- lymphatic flow disorders, including swelling of unclear origin near the area where the support will be applied.

### Safety notes:

- Product is for single patient use only.
- Indications, daily duration of use and period of application are generally determined by the physician.
- Ensure that the support is applied properly and fits correctly.