GAMINIO NAUDOJIMO INSTRUKCIJA

USER'S MANUAL

KAKLO ĮTVARAS KR 0-16 / CERVICAL ORTHOSIS KR 0-16

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Recommended for spinal column and neck area traumas, such as broken vertebra, severe bruising, dislocation and muscle and tendon strains. It should be used as a first aid means of immobilization.

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Size. In order the product performs its functions effectively correct size has to be chosen (see table below). Orthosis should not slip down, strangle or cause any other unfavourable sensations.

	XS	S	M	L
cm	26-31	31-34	34-37	37-40

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Recommendations:

Consult your doctor on the expediency of wearing the orthosis and the recommended time of wearing it. Do not wear the orthosis if you notice any skin sensitiveness.

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It is recommended to clean the product by alcohol means (inside) and water (outside). If necessary use mild laundry detergents. Dry at ambient temperature away from heating appliances and direct sun. Keep away from fire and heat. If the product is fitted with a Velcro, it should be washed and dried closed.

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The warranty is provided in accordance with the procedure established by the laws. The warranty obligations shall be observed provided that during the said period the product is used for its direct purpose and its construction is not modified.

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Thank you for choosing our product. We trust you'll feel better.

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