GAMINIO NAUDOJIMO INSTRUKCIJA

USER'S MANUAL

JUOSMENS-KRYŽKAULIO ĮTVARAI KR 0-17-3 / KR 0-17-4

LUMBOSACRAL ORTHOSIS KR 0-17-3 / KR 0-17-4

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Anatomically formed orthoses optimally compresses and supports the lumbar section of spine by lateral fixers of different rigidity. Silicone pellote massages the lumbar section of spine. If necessary, the pellote can be removed.

Recommended in cases of pain in the lumbar section, radiculitis, strain of muscles and ligaments in the lumbar section as well as to protect the lumbar section.

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Size. In order the product performs its functions effectively correct size has to be chosen (see Table below). Orthose should not slip down, strangle or cause any other unfavourable sensations.

	S	M	L	XL	XXL	3XL
cm	70-80	80-90	90-100	100-110	110-120	120-130

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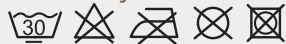
Recommendations:

The therapist shall be consulted regarding the expedience and duration of wearing the product in case of: skin solidity damages in the respective area of the body, bloodstream disorders, heart deficiency. The waist area shall not be covered with oil, lotions or other greasy preparations before putting on the support. Do not wear the support in case of any allergic skin changes or other unpleasant sensations.

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The warranty is provided in accordance with the procedure established by the laws. The warranty obligations shall be observed provided that during the said period the product is used for its direct purpose and its construction is not modified.

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Thank you for choosing our product. We trust you'll feel better.

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