## KULKŠNIES-PĖDOS ĮTVARAS KT 0-4-3 ANKLE-FOOT ORTHOSIS KT 0-4-3

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**Recommended wear:** in cases of swelling caused by arthrosis and arthritis, tendinomyopathy, weak tarsus ligaments, damage and inflammation of the tarsus knee ligaments, tendons and muscles as well as for trauma prevention.

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**Size.** If you want the product to perform its functions effectively, it is necessary to choose the right size (see Table below). A correctly chosen orthosis should not slip down, strangle or cause any other unfavourable sensations.

	XS	S	M	L	XL	XXL
cm	17-19	19-21	21-23	23-25	25-27	27-29

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**Recommendations:** Consult your therapist on the expedience and duration of wearing the product, especially in cases of skin damages in the area of tarsus joint, bloodstream disorders or intensive swelling in the area of tarsus joint, diabetic foot and diabetes mellitus.

The area of tarsus shall not be covered with oil, lotions or other greasy preparations before putting on the orthosis. Do not wear the orthosis in case of any allergic skin changes or other unpleasant sensations.

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**The warranty** is provided in accordance with the procedure established by the laws. The warranty obligations shall be observed provided that during the said period the product is used for its direct purpose and its construction is not modified.

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**Thank you for choosing our product. We trust you'll feel better.**

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