## KELIO ĮTVARAS KT 3-5-0 / KNEE ORTHOSIS KT 3-5-0

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**Recommended** during the knee treatment of inflammation, strain of ligaments and tendons, hemarthrosis and gonarthrosis; to provide first aid; during recovery period after knee joint surgery. Orthosis made of rigid, breathable fabric with integrated fasteners aluminum.

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**Size.** If you want the product to perform its functions effectively, it is necessary to choose the right size (see Table below). A correctly chosen orthosis should not slip down, strangle or cause any other unfavourable sensations.

Dydis/ Size	XS	S	M	L	XL
a, cm	38-43	44-48	49-53	54-58	59-63
b, cm	28-32	33-36	37-40	41-44	45-48

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**Recommendations:** Consult your doctor on the expediency of wearing the orthosis and the recommended time of wearing it. If it necessary, the product can be in addition disband and adapted to the anatomical shape of the leg.

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It is recommended to hand-wash the orthosis in water not warmer than 30°C. Use mild laundry detergents. Dry at ambient temperature away from heating appliances and direct sun. Keep away from fire and heat. If the product is fitted with a Velcro, it should be washed and dried closed.

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**The warranty** is provided in accordance with the procedure established by the laws. The warranty obligations shall be observed provided that during the said period the product is used for its direct purpose and its construction is not modified.

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**Thank you for choosing our product. We trust you'll feel better.**

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