GAMINIO NAUDOJIMO INSTRUKCIJA USER'S MANUAL

KELIO ĮTVARAS KT 3-8-5 / KNEE ORTHOSIS KT 3-8-5

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Flexible lateral splints and fastening straps stabilize knee joint. The orthosis stabilizes kneecap. Used in cases of strain, rupture, trauma of knee joint ligaments, during rehabilitation period after knee joint operations.

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Composition: 75% elastic polyurethane, 12.5% polyamide, 12.5% viscose.

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Size. If you want the product to perform its functions effectively, it is necessary to choose the right size (see Table below). A correctly chosen orthosis should not slip down, strangle or cause any other unfavourable sensations.

	XS	S	M	L	XL
cm	30-33	33-36	36-40	40-44	44-46

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Directions for use:

1. Place the orthosis on knee joint position where the split with the pad fix the kneecap.
2. Fasten the size-adjusting strips.
3. Fasten the reinforcing straps.

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Recommendations:

Consult your doctor on the expediency of wearing the orthosis and the recommended time of wearing it. Do not wear the orthosis if you notice any skin sensitiveness.

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It is recommended to hand-wash the orthosis in water not warmer than 30°C. Use mild laundry detergents. Be careful to take out all hard details/parts before washing. If the product is fitted with velcro, it should be washed and dried closed. If you wash in a washing machine, place the orthosis into a special bag to prevent the Velcro from spoiling other clothes.

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The warranty is provided in accordance with the procedure established by the laws. The warranty obligations shall be observed provided that during the said period the product is used for its direct purpose and its construction is not modified.

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Thank you for choosing our product. We trust you'll feel better.

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