# GAMINIO NAUDOJIMO INSTRUKCIJA USER'S MANUAL

## KELIO ĮTVARAS KT 3-9-1 / KNEE ORTHOSIS KT 3-9-1

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Anatomically formed orthosis optimally compresses and slightly supports the knee joint by lateral flexible fixers. Silicone ring stabilizes the knee patella to be in the right position.

**Recommended wear:** in cases of knee pain, diseases of joint cartilages, faulty position of the knee patella, strain of the knee joint ligaments and muscles, arthrosis, osteoarthritis, synovitis, chondritis, patella osteochondropathies of various origins, inflammation of tendons and joint bursae of knee joint section, osteochondropathy of tibial tuberosity, instability of patella, inflammation in the patellar ligament as well as for trauma prevention.

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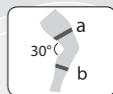
**Size.** In order the product performs its functions effectively correct size has to be chosen (see Table below). Orthosis should not slip down, strangle or cause any other unfavourable sensations.

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**When taking measurements**, the knee should be in 30° flexion, thigh at least 14 cm above middle of knee; calf – at least 12 cm below middle of knee.


	XS	S	M	L	XL	XXL
Apimtis / circumference a (viršuje / above)	38-41	41-44	44-47	47-50	50-53	53-56
Apimtis / circumference b (apačioje / below)	28-31	31-34	34-37	37-40	40-43	43-46

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**Recommendations:** The therapist shall be consulted regarding the expedience and duration of wearing the product in case of: skin solidity damages in the area of knee joint, varicose leg veins, bloodstream disorders, swelling of unknown origin. The area of knee joint shall not be covered with oil, lotions or other greasy preparations before putting on the orthosis. Do not wear the orthosis in case of any allergic skin changes or other unpleasant sensations.

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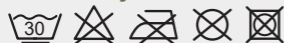**The warranty** is provided in accordance with the procedure established by the laws. The warranty obligations shall be observed provided that during the said period the product is used for its direct purpose and its construction is not modified.

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**Thank you for choosing our product. We trust you'll feel better.**

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