GAMINIO NAUDOJIMO INSTRUKCIJA USER'S MANUAL

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Recommended for immobilization of the arm after ligaments or injuries, tingling or muscle fractures; at nerve, muscle, tendon and shoulder joint inflammation, paralysis; during the rehabilitation period after operations, shoulder dislocation.

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Size. If you want the product to perform its functions effectively, it is necessary to choose the right size (see Table below). A correctly chosen orthosis should not slip down, strangle or cause any other unfavourable sensations.

	XXS	XS	S	M
cm	25-30	30-35	35-40	40-45

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Directions for use:

1) Place the support on the folded arm. Apply the lower band to the waist, and screw its end through the loop on the inside of the arm holder and secure it with a buckle to the tape.

2) Thread the upper band from the abdominal diagonal through the back and wrap the neck. Pull the free end through the loop at the top of the arm holder and secure the swath to the height so that the hand is locked in the required position.

Both bands can be shortened (cut off) according to need, after removing the buckles.

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Recommendations:

Consult your doctor on the expediency of wearing the orthosis and the recommended time of wearing it. Do not wear the orthosis if you notice any skin sensitisations.

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The warranty is provided in accordance with the procedure established by the laws. The warranty obligations shall be observed provided that during the said period the product is used for its direct purpose and its construction is not modified.

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Thank you for choosing our product. We trust you'll feel better.

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