## RIEŠO - PLAŠTAKOS ĮTVARAS RT 1-8-2 WRIST-HAND ORTHOSIS RT 1-8-2

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**Recommended** for use in the rehabilitation period following a broken radius. It can also be used for the treatment of inflammation, bruising, tendon and ligament strains of wrist joint or hand.

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**Size.** If you want the product to perform its functions effectively, it is necessary to choose the right size (see Table below). A correctly chosen orthosis should not slip down, strangle or cause any other unfavourable sensations.

K/D	L/R	XS	S	M	L
	cm	12-14	14-16	16-18	18-20

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### Recommendations:

Consult your doctor on the expediency of wearing the orthosis and the recommended time of wearing it. Do not wear the orthosis if you notice any skin sensitiveness.

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It is recommended to wash the orthosis in water not warmer than 30°C. Use mild laundry detergents. Dry at ambient temperature away from heating appliances and direct sun. Keep away from fire and heat. If the product is fitted with a Velcro, it should be washed and dried closed. If you wash in a washing machine, place the orthosis into a special bag to prevent the Velcro from spoiling other clothes.

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**The warranty** is provided in accordance with the procedure established by the laws. The warranty obligations shall be observed provided that during the said period the product is used for its direct purpose and its construction is not modified.

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**Thank you for choosing our product. We trust you'll feel better.**

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