## RIEŠO – PLAŠTAKOS ĮTVARAS RT 1-8-6 WRIST – HAND ORTHOSIS RT 1-8-6

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An anatomically formed wrist - hand orthosis with silicone pelote an additional fixation control elastic straps, optimally compresses and easily consolidates your wrist and hand in the correct position.

**Recommended** during rehabilitation after wrist joint injuries, after forearm fracture.

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**Size.** In order the product performs its functions effectively correct size has to be chosen (see Table below). Orthosis should not slip down, strangle or cause any other unfavourable sensations.

K/D	XS	S	M	L	XL	
L/R	cm	12-14	14-16	16-18	18-20	20-22

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**Recommendations:** consult your doctor on the expediency of wearing the orthosis and the recommended time of wearing it. Do not wear the orthosis if you notice any skin sensitivities. The area of wrist-hand joint shall not be covered with oil, lotions or other greasy preparations before putting on the support.

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It is recommended to wash the orthosis in water not warmer than 30°C. Use mild laundry detergents. Dry at ambient temperature away from heating appliances and direct sun. Keep away from fire and heat. If the product is fitted with a Velcro, it should be washed and dried closed. If you wash in a washing machine, place the orthosis into a special bag to prevent the Velcro from spoiling other clothes.

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**The warranty** is provided in accordance with the procedure established by the laws. The warranty obligations shall be observed provided that during the said period the product is used for its direct purpose and its construction is not modified.

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**Thank you for choosing our product. We trust you'll feel better.**

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