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The orthosis warms elbow joint. Used in cases of inflammation of elbow joint muscles and tendons, bursitis.

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Composition: 75% elastic polyurethane, 12.5% polyamide, 12.5% viscose.

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Size. In order the product performs its functions effectively correct size has to be chosen (see Table below). Orthosis should not slip down, strangle or cause any other unfavourable sensations.

	XS	S	M	L	XL
cm	23-25	25-27	27-30	30-33	33-36

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Directions for use:

Place the orthosis on the elbow joint and fasten the size-adjusting strips (Picture a). Intercross the supporting straps on the inner side of the arm below the bend of the elbow (Picture b) and fasten them at the bottom of the orthosis (Picture c).

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Recommendations: Consult your doctor on the expediency of wearing the orthosis and the recommended time of wearing it. The orthosis is designed for sports activities, thus it is not recommended to wear it all day long. Do not wear the orthosis if you notice any skin sensitiveness.

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The warranty is provided in accordance with the procedure established by the laws. The warranty obligations shall be observed provided that during the said period the product is used for its direct purpose and its construction is not modified.

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Thank you for choosing our product. We trust you'll feel better.

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