GAMINIO NAUDOJIMO INSTRUKCIJA USER'S MANUAL

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Anatomically formed orthose slightly supports the elbow joint. Silicone pelletes help to equally distribute load on surrounding soft tissues. The orthose will always remain in the proper place (will not slip off) as these pelletes repeat the anatomy of joint and fix the orthose on humerus condyles.

Recommended wear: in cases of epicondylitis, swelling caused by arthrosis/arthritis, after traumas and surgeries as well as for trauma prevention.

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Size. In order the product performs its functions effectively correct size has to be chosen (see Table below). Orthose should not slip down, strangle or cause any other unfavourable sensations.

	XS	S	M	L	XL	XXL
cm	19-21	21-23	23-25	25-27	27-29	29-31

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Recommendations:

The therapist shall be consulted regarding the expedience and duration of wearing the product in case of: skin solidity damages in the area of elbow joint, bloodstream disorders below the elbow joint, intensive swelling in the forearm and brachium area.

The area of elbow joint shall not be covered with oil, lotions or other greasy preparations before putting on the support. Do not wear the support in case of any allergic skin changes or other unpleasant sensations.

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The warranty is provided in accordance with the procedure established by the laws. The warranty obligations shall be observed provided that during the said period the product is used for its direct purpose and its construction is not modified.

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Thank you for choosing our product. We trust you'll feel better.

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